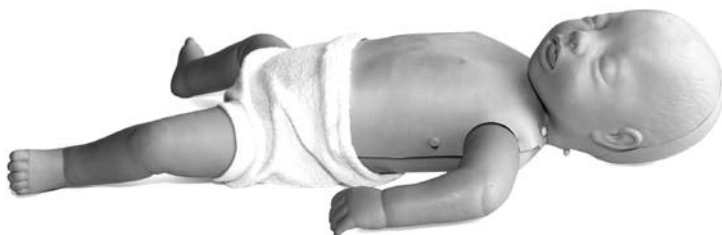
Resusci Baby

ENGLISH	Directions for Use
DEUTSCH	Gebrauchsanweisung
FRANÇAIS	Mode d'emploi
ESPAÑOL	Instrucciones de utilización
ITALIANO	Istruzioni per l'Uso
NEDERLANDS	Handleiding
NORSK	Bruksanvisning
SVENSKA	Bruksanvisning
SUOMI	Käyttöohje

Your new Resusci Baby

Resusci Baby has been designed to serve as a realistic training aid for teaching infant CPR skills. Some helpful features included in the manikin for this purpose are:

- Natural obstruction of the airway
- Movable jaw
- Hyperextension of the head causes reclosure of the airway
- Human-like compliance for ventilations and compressions
- Chest rise with ventilations
- Realistic landmarks for compression point location
- Brachial pulse
- Indication of too fast inflation (with Skillguide)

Unpacking and Assembly

Resusci Baby is provided assembled and ready to use.

Some additional supplies are provided for your convenience. Included with your Resusci Baby manikin (cat. No. 14 00 10) are:

- 2 extra Resusci Baby Faces
- 5 extra Resusci Baby Airways
- Pulse bulb and tubing
- Directions for use
- Sanitation kit

Skillguide Signal Box

Consistent and objective feedback on performance during practice helps develop and reinforce skills. The Skillguide Signal Box provides feedback to the instructor and student to support the student's initial skill development by means of a color light display.

If you purchased the Resusci Baby with Skillguide (cat. No. 14 00 11) you will also have received:

- Skillguide signal Box
- Four 1.5 V alkaline batteries, size C

To connect and use the Skillguide Signal Box:

1. Plug the Skillguide cable into the connector on the right side of the manikin.

2. Remove the paper tab coming from the battery compartment on the backside of the signal box.
3. Press the "On" button to begin practice.

The Skillguide will turn "off" automatically if no activity takes place for 5 minutes.

Note: If the Skillguide is turned "on" and the red signal for finger position comes on, the Skillguide is not completely plugged in to the manikin.

Power Supply

Four 1.5 V alkaline batteries, size C.

This will provide approximately 125 hours of use in the classroom. This will vary depending on use patterns and brand of batteries.

Changing batteries

When replacing worn out batteries in your Skillguide Signal Box, we encourage you to replace them with alkaline batteries.

To replace batteries:

1. Place your thumb on the marked area next to the arrow.
2. Push firmly in the direction of the arrow to slide open the battery cover.
3. Remove and discard old batteries.
4. Replace new batteries as indicated on battery cover.
5. Align battery cover with grooves in battery compartment.
6. Slide cover and snap closed.

Connection cable

This is permanently connected to the Skillguide and has an 8 pin male connector which is to be connected to the manikin. The connector has been molded for proper orientation prior to plugging it into the manikin's side.


Caution:

Make sure that the Skillguide is unplugged from manikin:

- when airway is changed, or
- when internal service is carried out

	Light	Indication	Measurement ^{1, 2}
Inflation volume:	green orange	correct too much	30 ml 50 ml
Compression depth:	green orange	correct too much	.5 in (13 mm) 1.0 in (20 mm)
Wrong finger position:	red	Pressure outside of acceptable area	
Too fast inflation	red		30 ml ≤ 0.27 sec

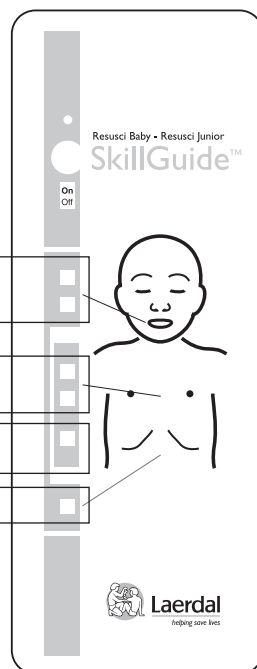1. Measurement tolerance in the manikin +/- 15% in normal room temperature
2. If manikin is used in laboratory tests, remove pants for accurate measurements

Sanitation

Sanitation of Manikin Faces, see separate sheet.


Resusci Baby Face.

We recommend you provide an individual Resusci Baby Face to each student. As an alternative you can use the same Resusci Baby Face for all students and clean the face between each student's use.

Using the Resusci Baby Face on an individual basis

Each student in the class should install and remove his or her manikin face.

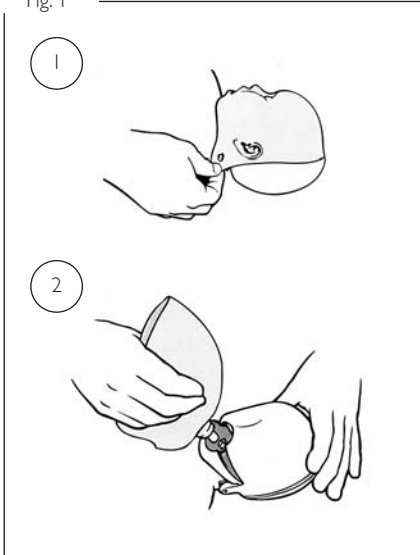
Fig. 1

To remove (Fig. 1)

1. Detach face from retainer at each ear.
2. Lift off face with coupling

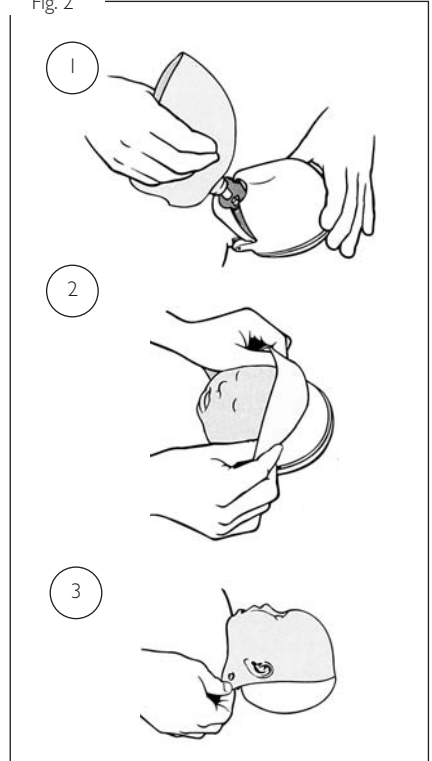Fig. 2


To install (Fig. 2)

1. Fold face upward to avoid direct finger contact with mouth and nose area. Push face coupling onto airway connector until fully seated.
2. Unfold face over head.
3. Fasten retainer at each ear

Disposable Resusci Baby Airway.

The Resusci Baby Airway is designed for disposal after class

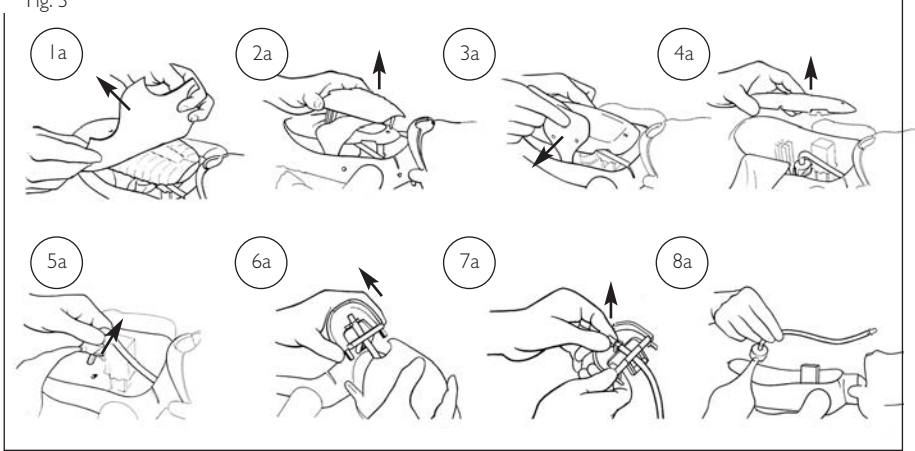Fig. 3


To remove (Fig. 3)


- 1a Remove chest skin
- 2a Lift off chest plate
- 3a Detach lung
- 4a Remove lung plate
- 5a Remove lung
- 6a Lift jaw from inner head
- 7a Disconnect airway from jaw
- 8a Remove and throw away used airway

To install (Fig. 4)

- 1b Insert airway through neck opening into inner head
- 2b Slide tubing through opening in jaw and snap in place
- 3b Install jaw in head
- 4b Pull valve and tubing into body. Push valve into hole in back plate
- 5b Replace lung plate
- 6b Align lung on guiding pins
- 7b Replace chest plate. Important: Press until upper pin snaps in place
- 8b Replace chest cover

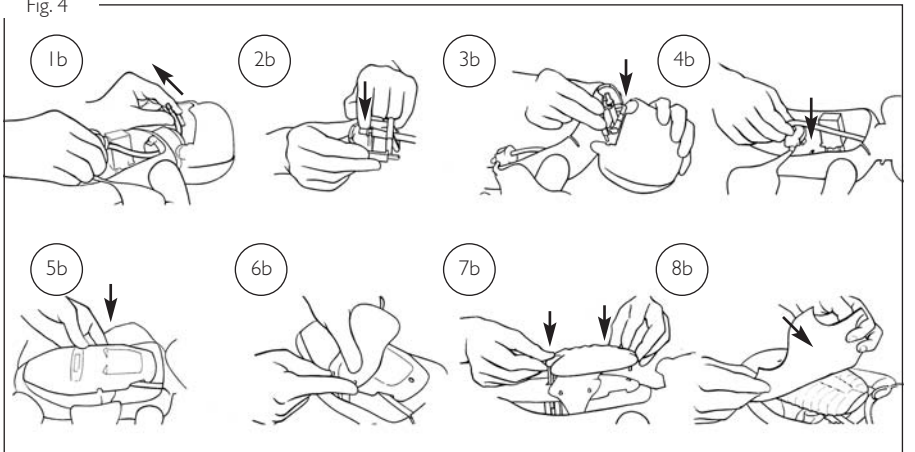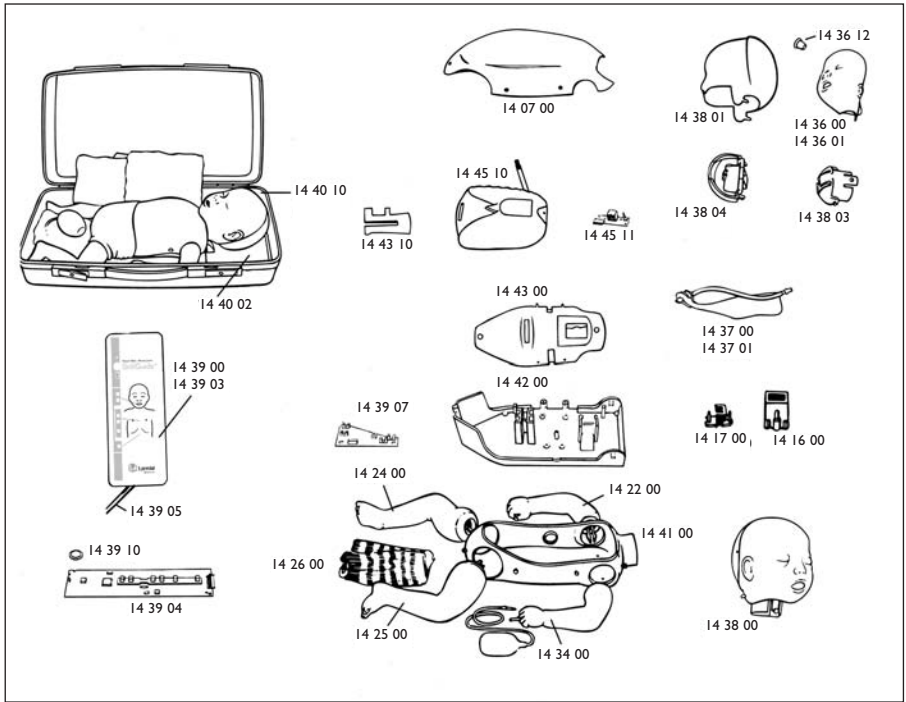Fig. 4

Parts

- 14 00 10 Resusci Baby, Basic
 14 00 11 Resusci Baby, with Skillguide

Common parts for both models

- 14 07 00 Torso skin
 14 16 00 Compression slider
 14 17 00 Compression spring assembly
 14 22 00 Right arm
 14 24 00 Right leg
 14 25 00 Left leg
 14 26 00 Pants
 14 34 00 Left arm w/pulse bulb & tubing
 14 36 12 Mask coupling (10)
 14 38 00 Head complete
 14 38 01 Inner head, complete

- 14 38 03 Inner neck
 14 38 04 Jaw
 14 40 10 Carrying case
 14 40 02 Tray for case
 14 41 00 Torso shell
 14 42 00 Back plate with screws
 14 43 00 Lung plate
 14 43 10 Ventilation slider
 14 49 00 Directions for Use

Parts for 14 00 11 only:

- 14 39 00 Skillguide, complete
 14 39 03 Battery cover
 14 39 04 Circuit board, Skillguide
 14 39 05 Cable
 14 39 07 Circuit board, vent/comp
 14 39 10 On/off cover
 14 45 10 Finger position kit w/circuit board and chest plate
 14 45 11 Circuit board side connection for foil-switch

Optional equipment and accessories

- 14 36 00 Resusci Baby Faces (6)
 14 36 01 Resusci Baby Faces (36)
 14 37 00 Disposable airways (24)
 14 37 01 Disposable Airways (96)
 15 24 00 Resusci Manikin Wipes (50)
 15 12 01 Manikin Face Shields (6 rolls of 36 each)
 15 42 30 Virkon Rapid Pack (10x5g tablets and bottle)
 15 42 31 Virkon tablets 10x5g

Update kit

- 14 45 00 Update kit Basic to Skillguide (not illustrated)

This product is in compliance with the essential requirements of Council Directive 89 / 336 / EEC, EMC Directive.

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