
Laerdal Paediatric Pocket Mask

ENGLISH	Directions for Use
FRANÇAIS	Mode d'emploi
ITALIANO	Istruzioni per l'Uso
ESPAÑOL	Instrucciones de utilización
NORSK	Bruksanvisning
NEDERLANDS	Handleiding
SVENSKA	Bruksanvisning
DANSK	Bruksanvisning
SUOMI	Käyttöohje
DEUTSCH	Gebrauchsanweisung
JAPANESE	取扱説明書
POLSKI	Instrukcja użytkowania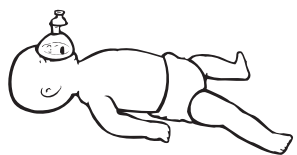

Laerdal
helping save lives

www.laerdal.com


1

2

3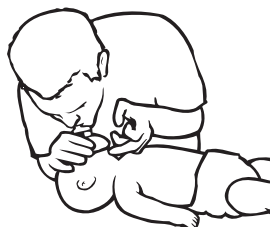

ENGLISH

PLEASE INSPECT AND VERIFY THAT THIS PRODUCT IS COMPLETE AND WITHOUT FLAWS BEFORE USE.

Applications

The Laerdal Paediatric Pocket Mask is a device designed for mouth-to-mask ventilation of a nonbreathing child or infant. This device should only be used by persons who have received adequate training.

1. Attach one-way valve to mask port by twisting the one-way valve until seated firmly (figure 1).
2. Open the patient's airway as taught in training.
3. Apply the mask over the patient's mouth or mouth and nose. If applying over the mouth only, as used for children, be sure to pinch the nostrils to avoid air leakage as taught in training (figure 2).
4. Seal as illustrated. Open airway. Blow slowly until chest rises (figure 3).
5. Remove your mouth. Allow patient to exhale.

Note: If patient vomits, remove mask and clear patient's airway. Clear mask assembly by shaking or sweeping out foreign material. Blow through valve to verify operation. Reapply mask to patient and continue ventilation.

Disassembly

1. Hold the mask with one hand and clamp the one-way valve between your thumb and two fingers of the other hand. Separate the two by twisting valve off.
2. Clean the one-way valve and mask as recommended if device is to be used by the same person after manikin training. Discard entire mask and one-way valve after patient use.

Cleaning

This mask is a single patient use device. **FOR TRAINING PURPOSES ONLY:** When used by the same person, the mask and valve can be cleaned and reused for training on manikins. **WARNING: DO NOT** reuse this product on a patient after it has been used for training purposes. Wash and scrub in warm soapy water. Rinse in clean water. Submerge for 10 min. in a 1:64 household bleach: water solution. Rinse again and allow to air dry.

Temperature limits

Operating: -10°C (14°F) to 40°C (104°F). Storage: -40°C (-40°F) to 50°C (122°F).

Expiratory resistance for the patient

Mask & one-way valve: Approximately 4.3 cm H₂O

Inspiratory resistance for the patient

Mask & one-way valve: Approximately 3.3 cm H₂O

Deadspace

90,5 ml

Performance data are nominal values only.

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