Resusci Anne

First Aid

Important Product Information

Resusci Baby

First Aid

Important Product Information

Laerdal

helping save lives

Notes

Important information about the product or its operation.

Care, Maintenance and Cleaning

For instructions on cleaning Resusci Anne and Resusci Baby faces, refer to the Cleaning Advice for Laerdal manikin faces insert. Keep the manikin clean and in a hygienic condition.

- Replace the airway assembly after every class, especially if mouth to mouth resuscitation is practiced.
- We recommend that you use a separate Laerdal Manikin Face for every student.
- If several students use one manikin face, thoroughly sanitize the manikin face after every use.
- Clean all skin parts regularly. Use warm soapy water or Laerdal manikin wipes.
- Use of Manikin Face Shields provides a clean barrier between your lips and the manikin face.

These can discolor the manikin:

- Pigments from lipstick and pens
- Latex gloves
- Using clothes other than Resusci Anne and Resusci Baby clothing.

Cautions

A Caution states a condition, hazard, or unsafe practice that can result in minor personal injury or damage to the manikin.

Caution Resusci Anne First Aid

- Use of automatic chest compression machines may damage the manikin.

Certification, Compliance and Labels

	Manufacturer
	Date of Manufacture
	Reference order number
	Warning / Caution symbol

Specifications

Operation temperature: 32°F to 104°F (0°C to +40°C)

Humidity: < 95% relative humidity

Storage temperature: 5°F to +122°F (-15°C to +50°C)

Dimensions Resusci Anne First Aid

Full body: 69.7" x 20.5" x 9.8" (177 cm x 52 cm x 25 cm)

Weight: 79.2 lbs (36 kg)

Dimensions Resusci Baby First Aid

Full body: 23.2" x 10.2" x 5.5" (59 cm x 26 cm x 13.5 cm)

Weight: 4.0 lbs (1.85 kg)

Warranty

The Laerdal Resusci Anne First Aid and Resusci Baby First Aid has a one-year limited Warranty. Refer to the Laerdal Global Warranty for terms and conditions.

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