

Infant IV Training Leg

Directions for Use



Table of Contents

Laerdal Recommends	3
Items Included	4
Skills Taught	4
Instructions for Use	4
Skin and Vein Replacement	4
Care and Maintenance	5
Replacement Parts	5

The Infant IV Training Leg (365-00101) is a full-size left infant leg, which realistically simulates that of a patient. It is specifically designed for training professionals in the practice of venipuncture.

As a result of quality manufacturing and the ease of replacing individual parts, this trainer should provide many sessions of training when reasonable care and maintenance is practiced.

Laerdal Recommends:

Venipuncture – Needle size 23 gauge

Instructor Notes:

Items Included:

- (1) Infant IV Training Leg
- (1) Laerdal Global Warranty Booklet
- (1) Directions for Use

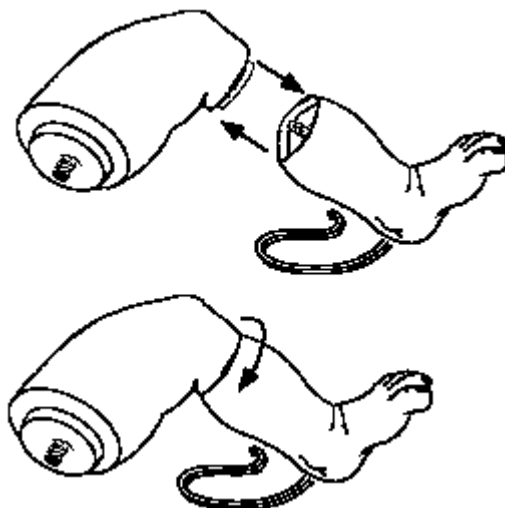
Skills Taught:

- Heel stick simulation
- Realistic flashback
- Fluid infusion
- Venipuncture in the medial and malleolus sites

Instructions for Use:

1. Fill a 12cc syringe with fluid. (Simulated blood concentrate is available from Laerdal Medical).
2. Attach fluid filled syringe to one latex vein.
3. Slowly inject fluid, allowing it to flow through leg and out other latex vein.
4. Using hemostats or a similar device, clamp latex vein through which fluid is flowing.

The IV leg is now ready to practice venipuncture.

**Skin and Vein Replacement:**

1. Slide used skin off leg mandrel.
2. Insert tubing through holes in new skin.
3. Pull tubing through holes while sliding new leg skin onto mandrel.
4. Screw foot using a clockwise motion onto upper leg until desired position is achieved.

Care and Maintenance:

1. Clean with mild soap and water; do not submerge in cleaning fluids or water.
2. Use only on clean surface. Avoid felt tipped markers, ink pens, acetone, iodine or other staining products and avoid placing the model on newsprint or inked lines of any kind.
3. To ensure longevity, each module should be cleaned after each training session and a general inspection should be conducted regularly.
4. Modules and all other parts should be drained and air-dried thoroughly before storage and disinfected when needed. After use of injection pads (*use water only*), accumulated water should be squeezed out. **Do not store wet foam pads in the skin.** To prevent mildew or mold, pads can be soaked in a mild solution of disinfectant and water or bleach and water.
5. Articulating parts will benefit from a light application of talcum powder prior to training sessions.
6. Store properly between teaching sessions.



Cautions and Warnings

This product contains **Natural Rubber** latex which may cause allergic reactions when in contact with humans.

REPLACEMENT PARTS:

The following replacement part numbers are for trainers with Texas Tan skin tone. Please contact Customer Service Representatives for replacement part numbers for trainers with other skin tone.

Product Number:

365-01050
365-01150
365-02850

Description:

Mandrel, ICD IV Leg
Sleeve, ICD IV Leg
Tubing, ICD IV Leg

© Laerdal Medical, Gatesville TX. All rights reserved.
1005315 rev C
August 30, 2006