



Replacement Instructions


- 1 Remove the manikin chest skin as shown below.

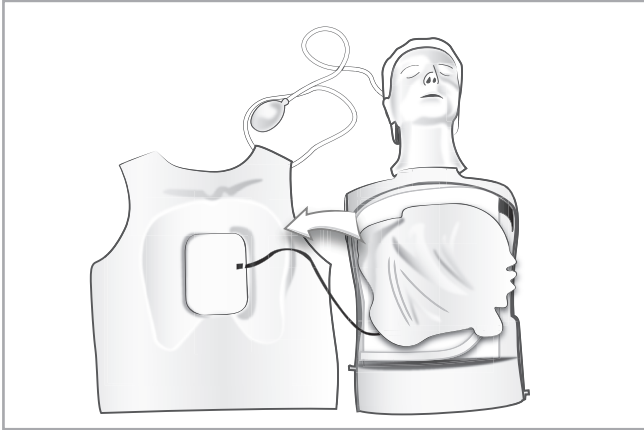


- 2 Lift and set aside the lung plate as shown below.

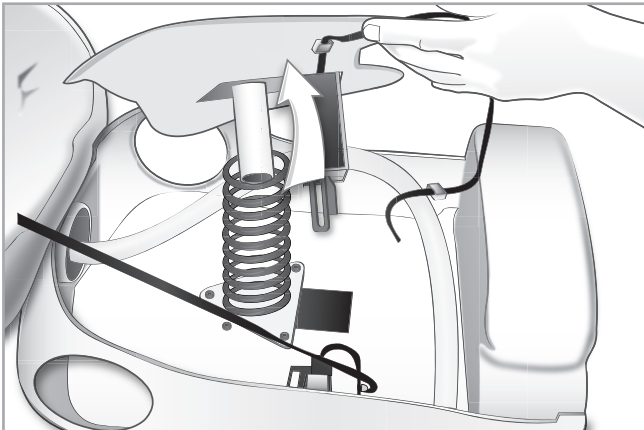


- 3 Remove the cable with strain relief from the lung plate and the side connection unit board as shown below.

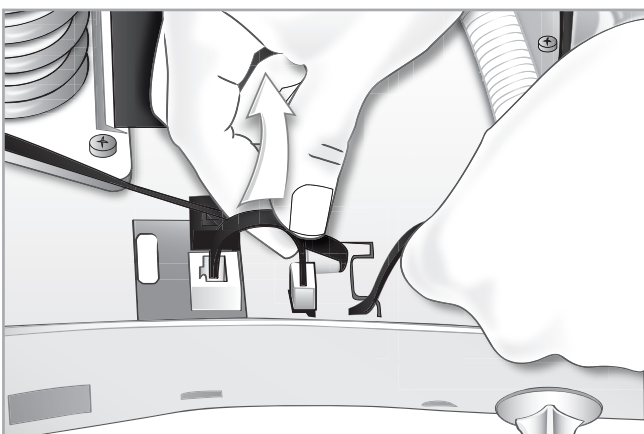


- 4 Remove the 4 retaining screws and discard the old sensor unit.

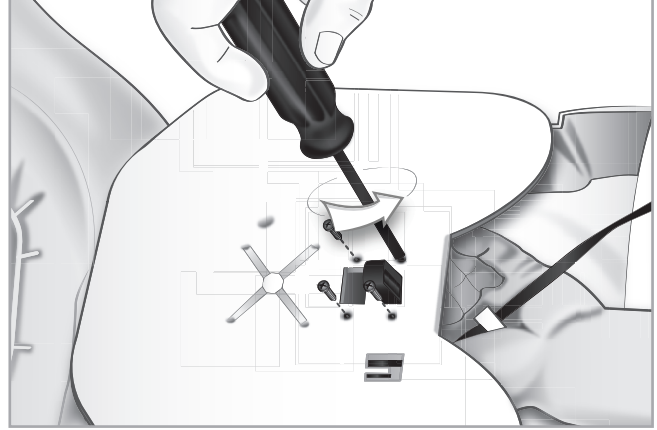

- 5 Align new sensor unit as shown below (only fits one way).

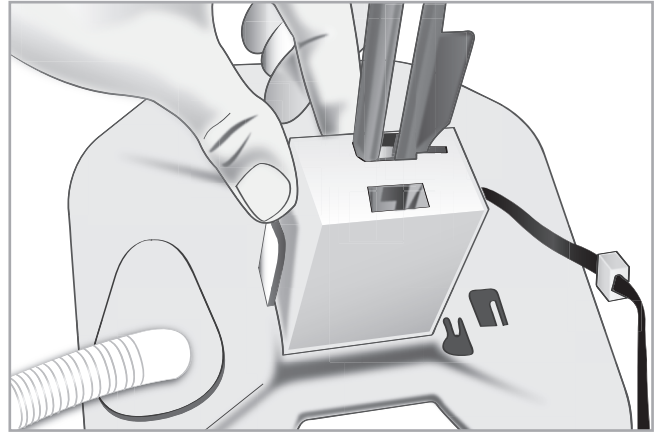

- 6 Install 4 retaining screws to the new sensor unit as shown below.

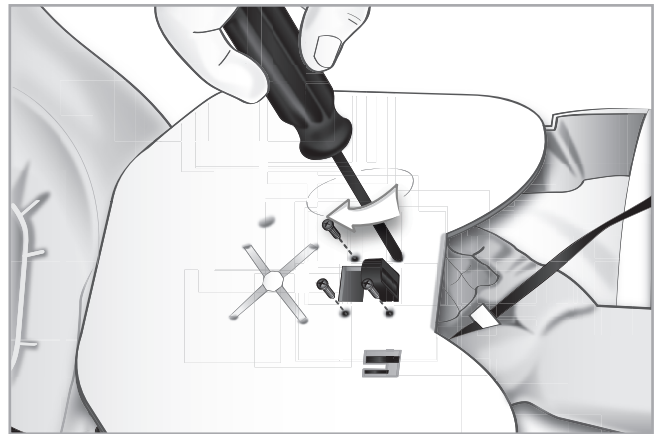

- 7 Route and reconnect sensor cables back to the lung plate and side connection unit board, as shown on step 3. Reinstall lung plate as shown in step 2 and re-attach manikin chest skin.



Laerdal
helping save lives