

Neonatal Resuscitation Scenarios - Upgrade instructions.


Instructions:

1. Uninstall Neonatal Resuscitation Scenarios
2. Install the upgraded 'Neonatal Resuscitation Scenarios' by following the setup wizard


Detailed Instructions:

1. Uninstall 'Neonatal Resuscitation Scenarios'


- Click Start -> Control Panel ->Add or Remove Programs
- Select 'Neonatal Resuscitation Scenarios' and click the Remove button.
- Click 'Next' when the Welcome window appears

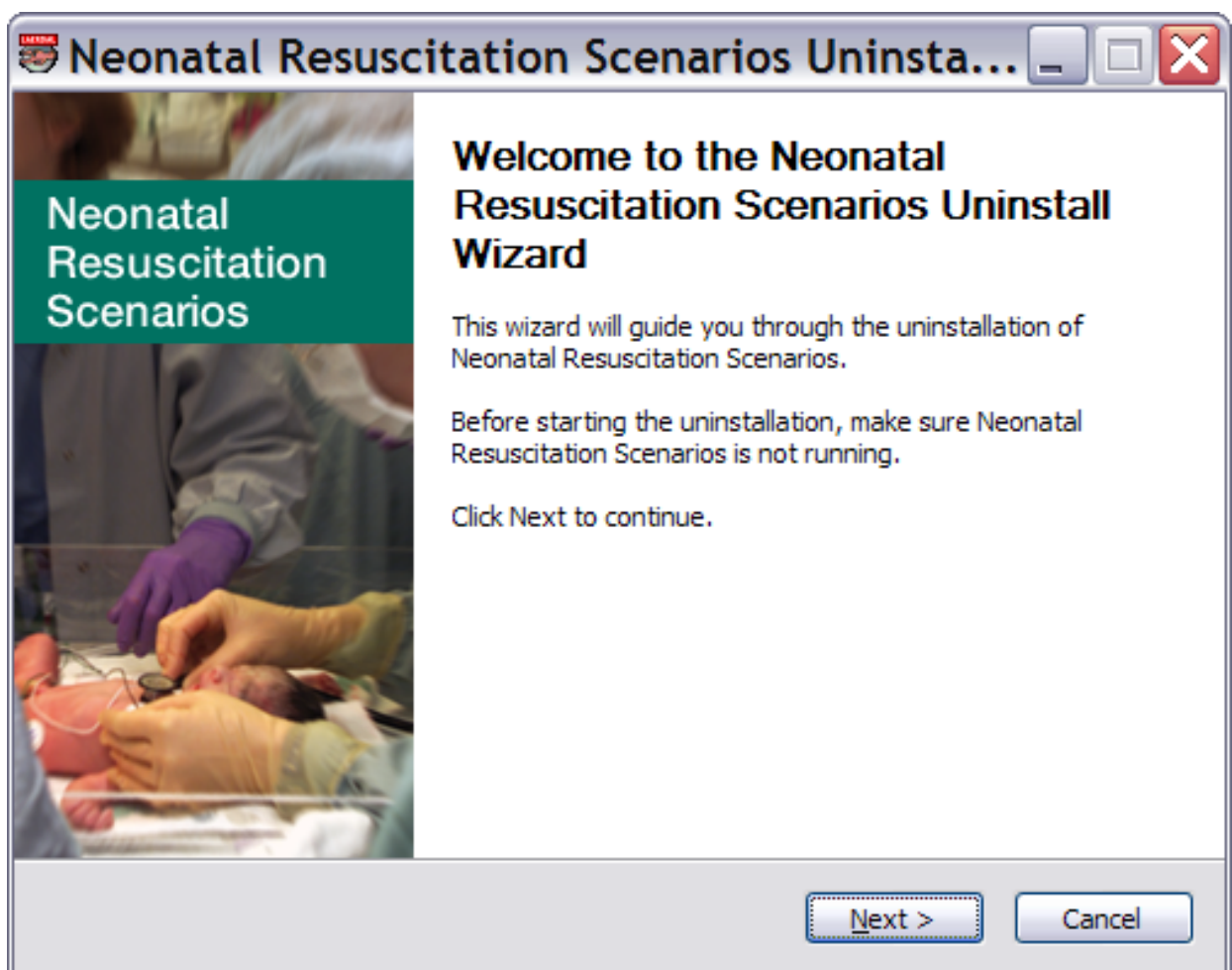

- The uninstall program is ready to - Click 'Uninstall'.

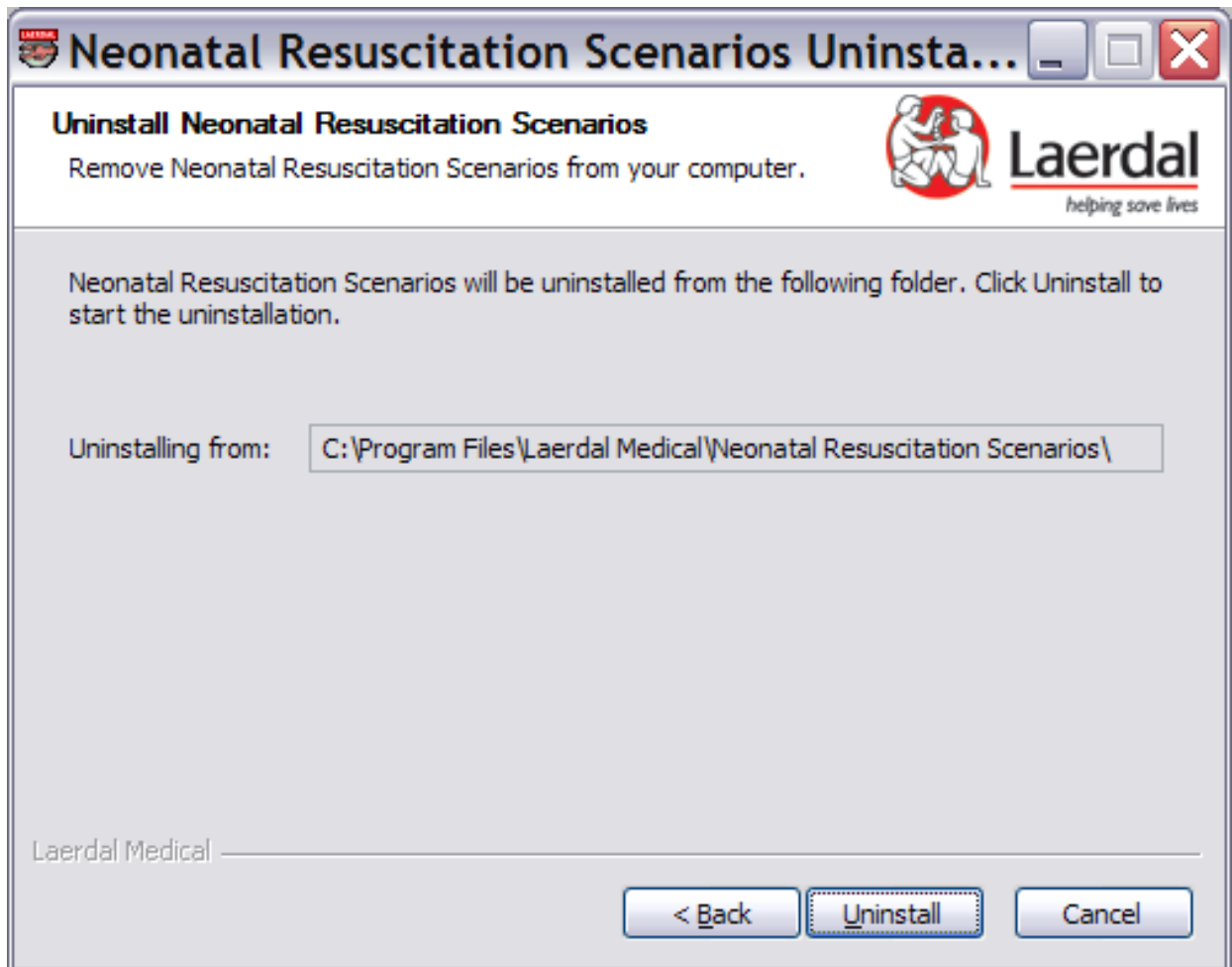


- Uninstall is complete - Click 'Finish' to close the program.

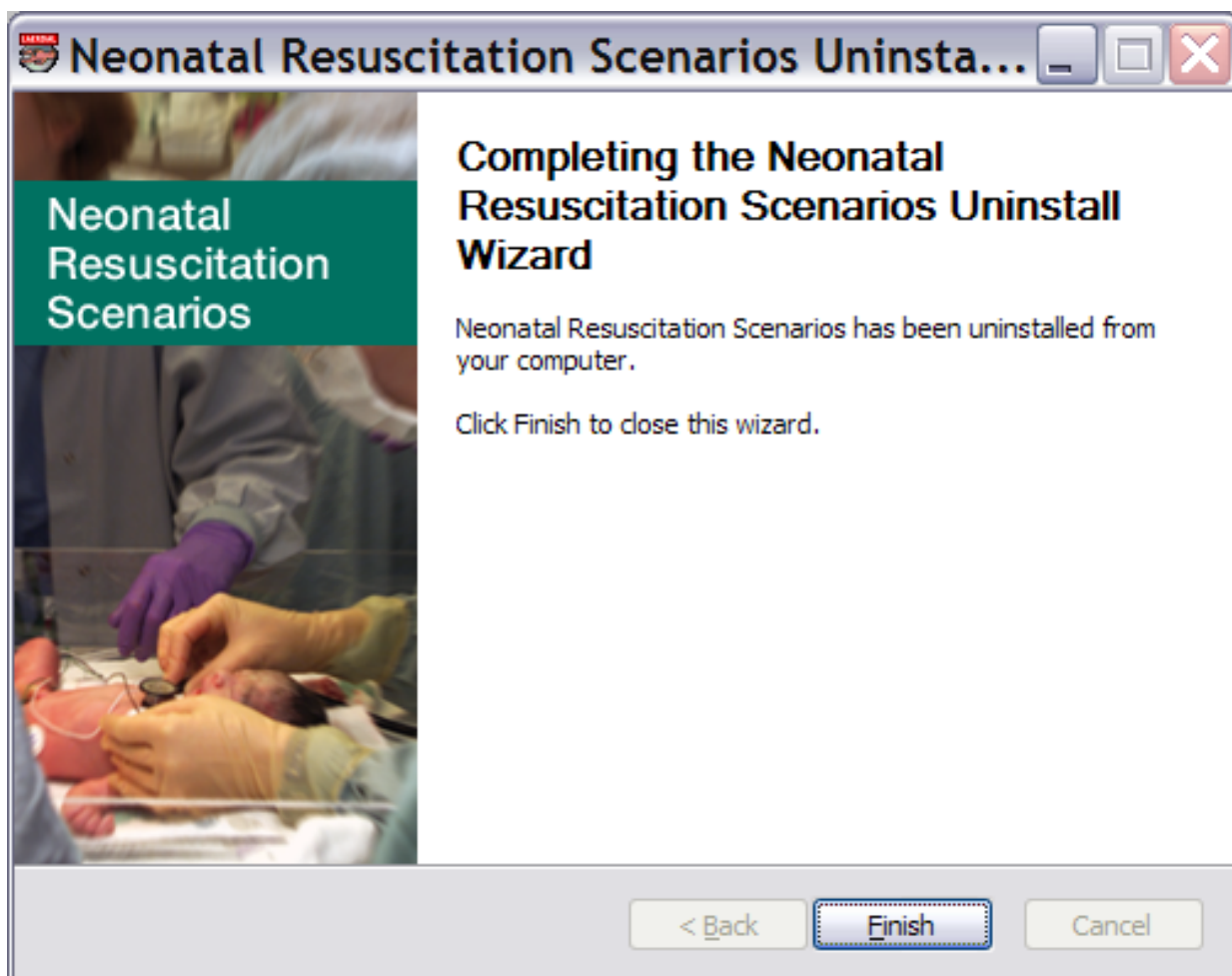


2. Install the upgraded 'Neonatal Resuscitation Scenarios'

- Download the 'Neonatal Resuscitation Scenarios' software from www.laerdal.com/download (click on SimNewB Advanced in the Product Downloads list)
- Unzip the downloaded file and launch setup.exe from the unzipped download folder.
- Click 'Next' when the Welcome window appears

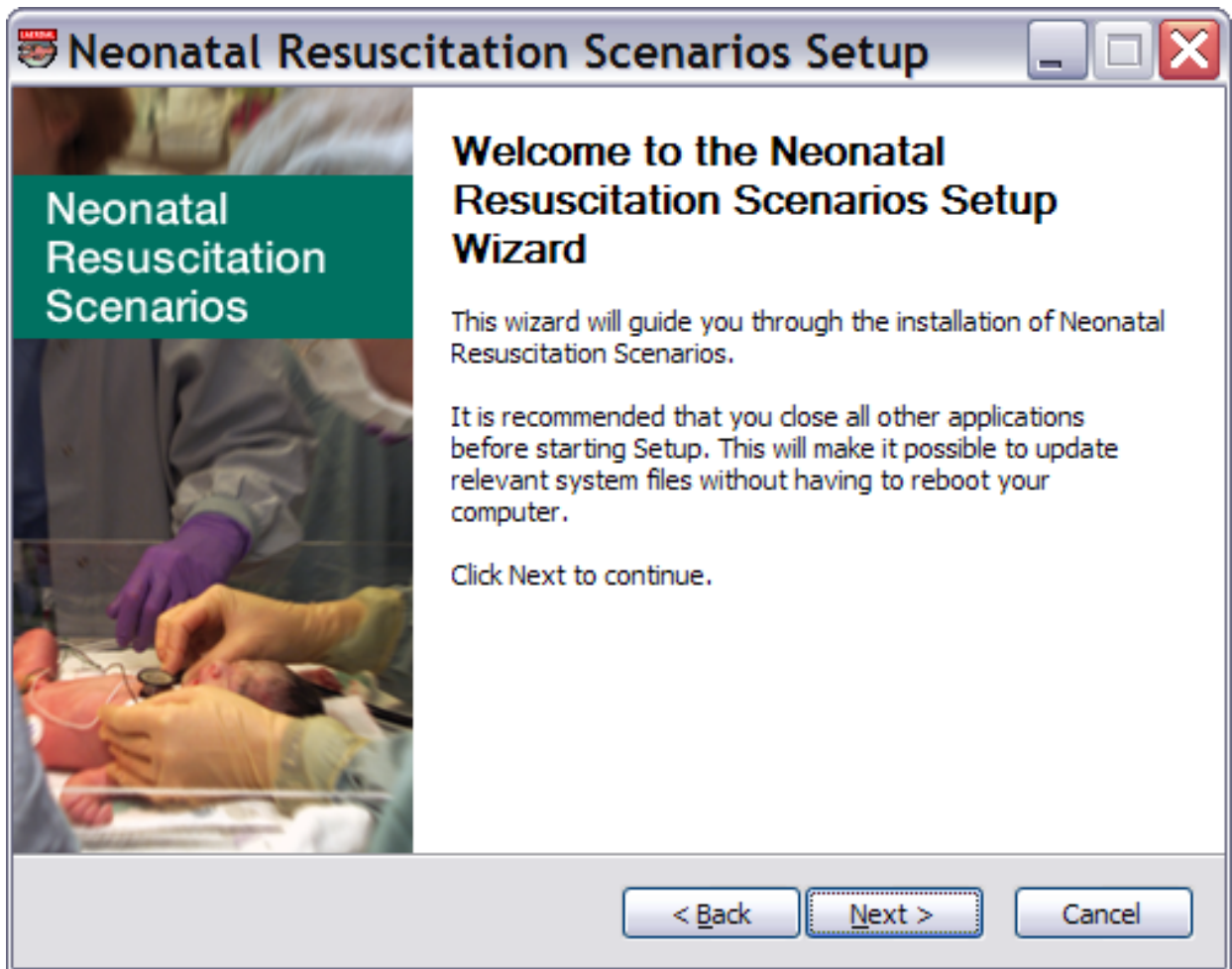


- Check the checkbox named 'I accept the terms of the License Agreement' and click 'Next'.

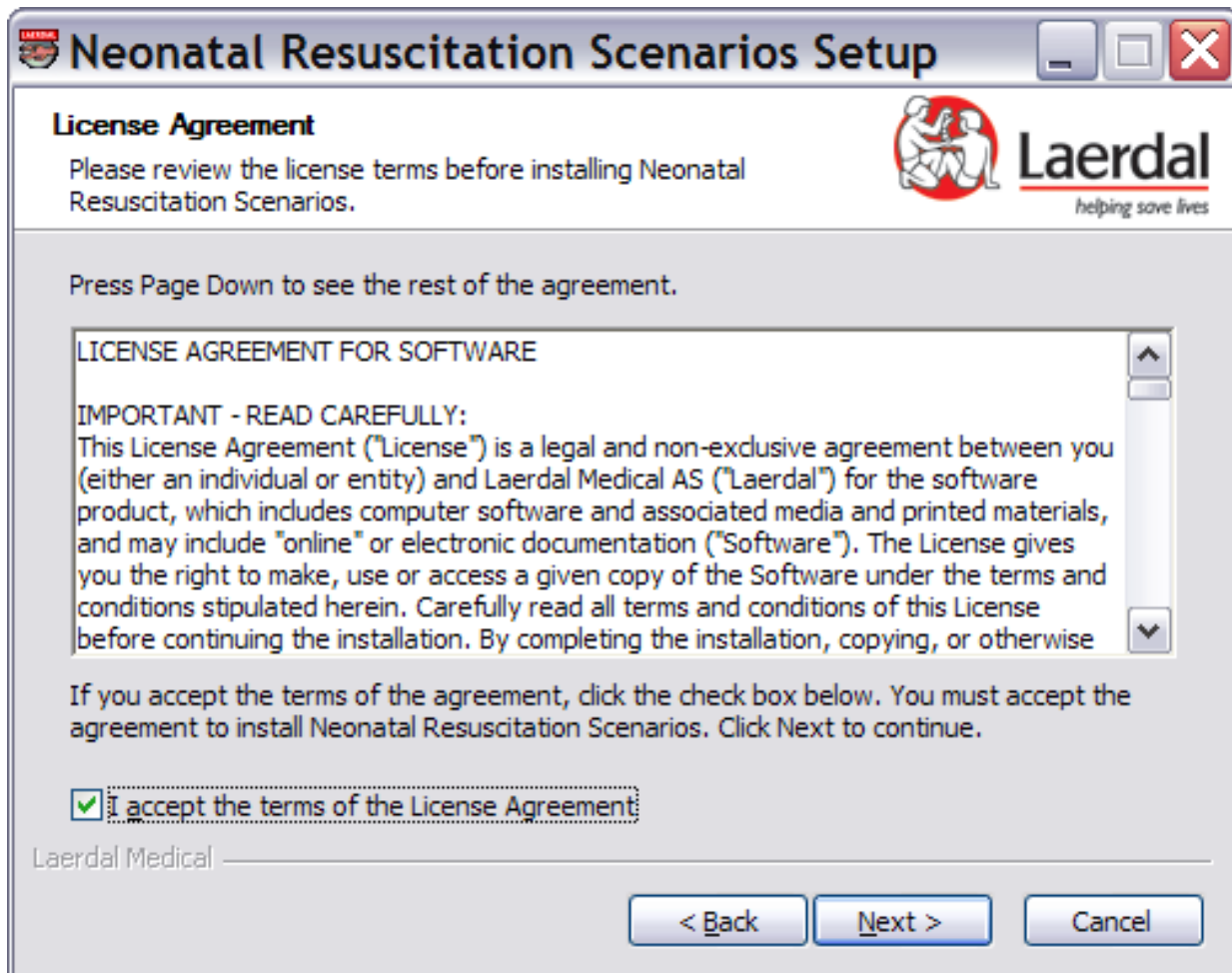


- Check the checkbox 'Replace folder for existing User Profiles' and click 'Next'. (This will make sure that all the existing users will get the upgraded versions of the scenarios.)


Note: This window will only appear if you have SimNewB 1.2 or earlier installed.

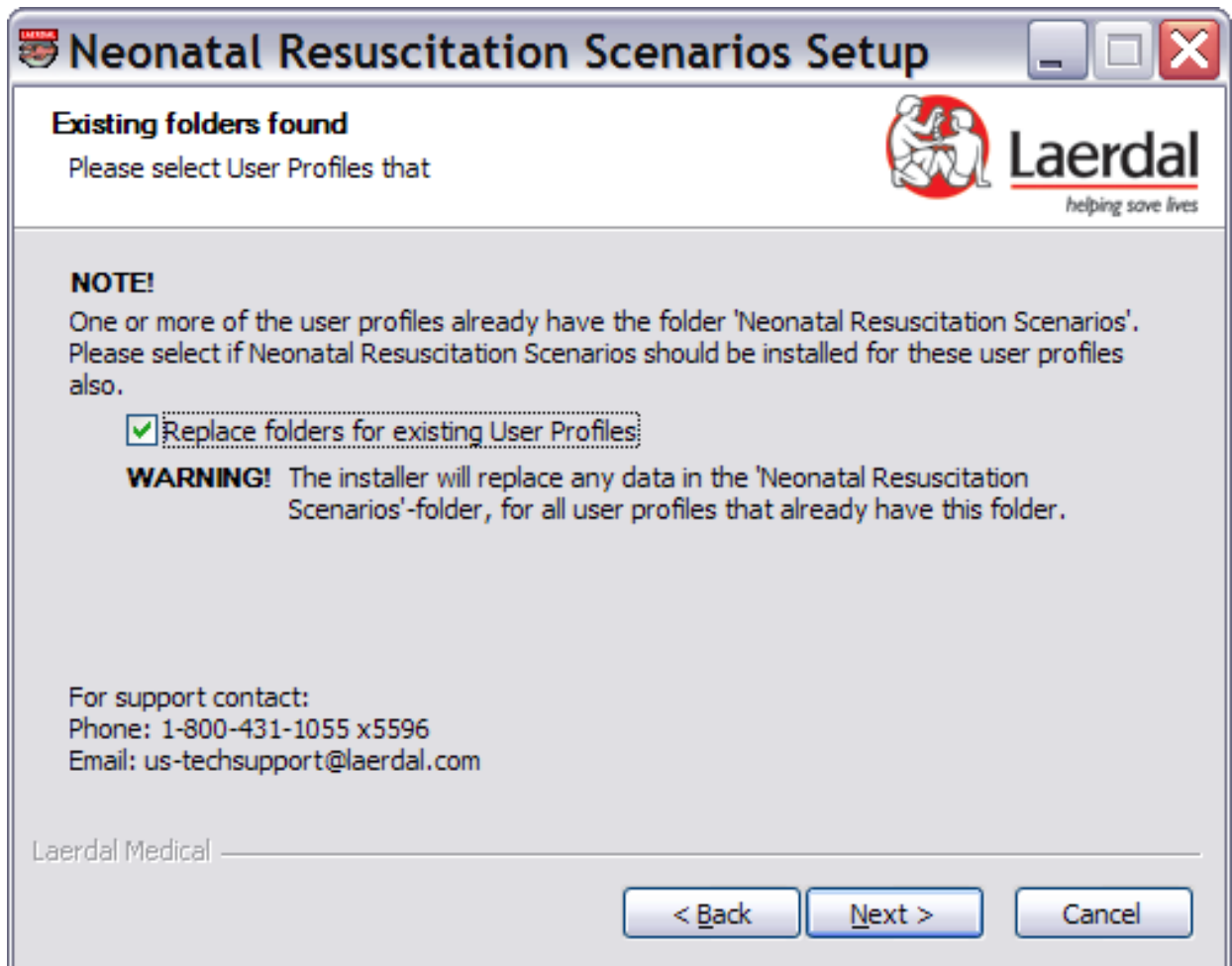

- Installation is ready to begin. Click 'Install' to start the installation.

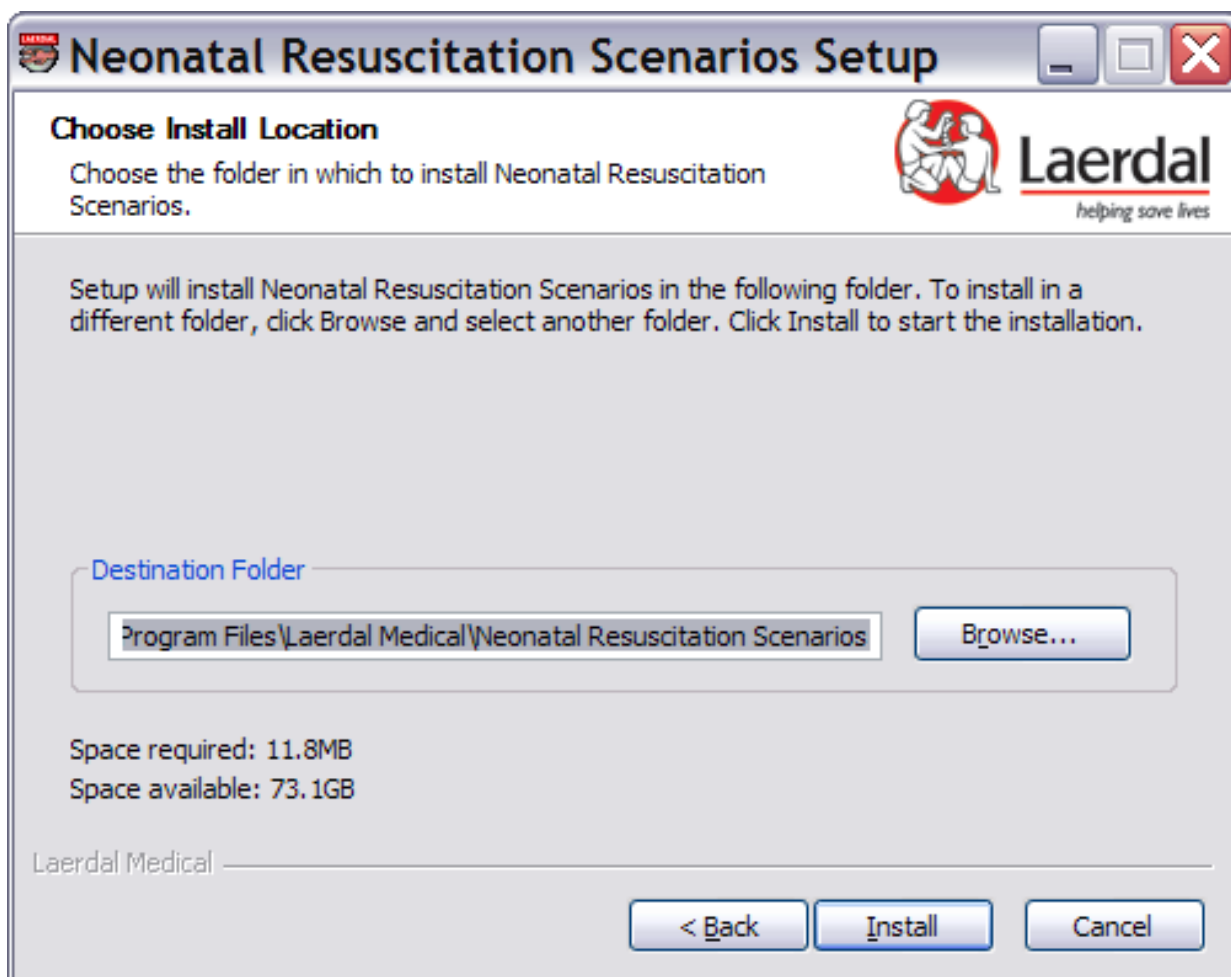

- When the installation is complete, click 'Finish' to end the setup program.

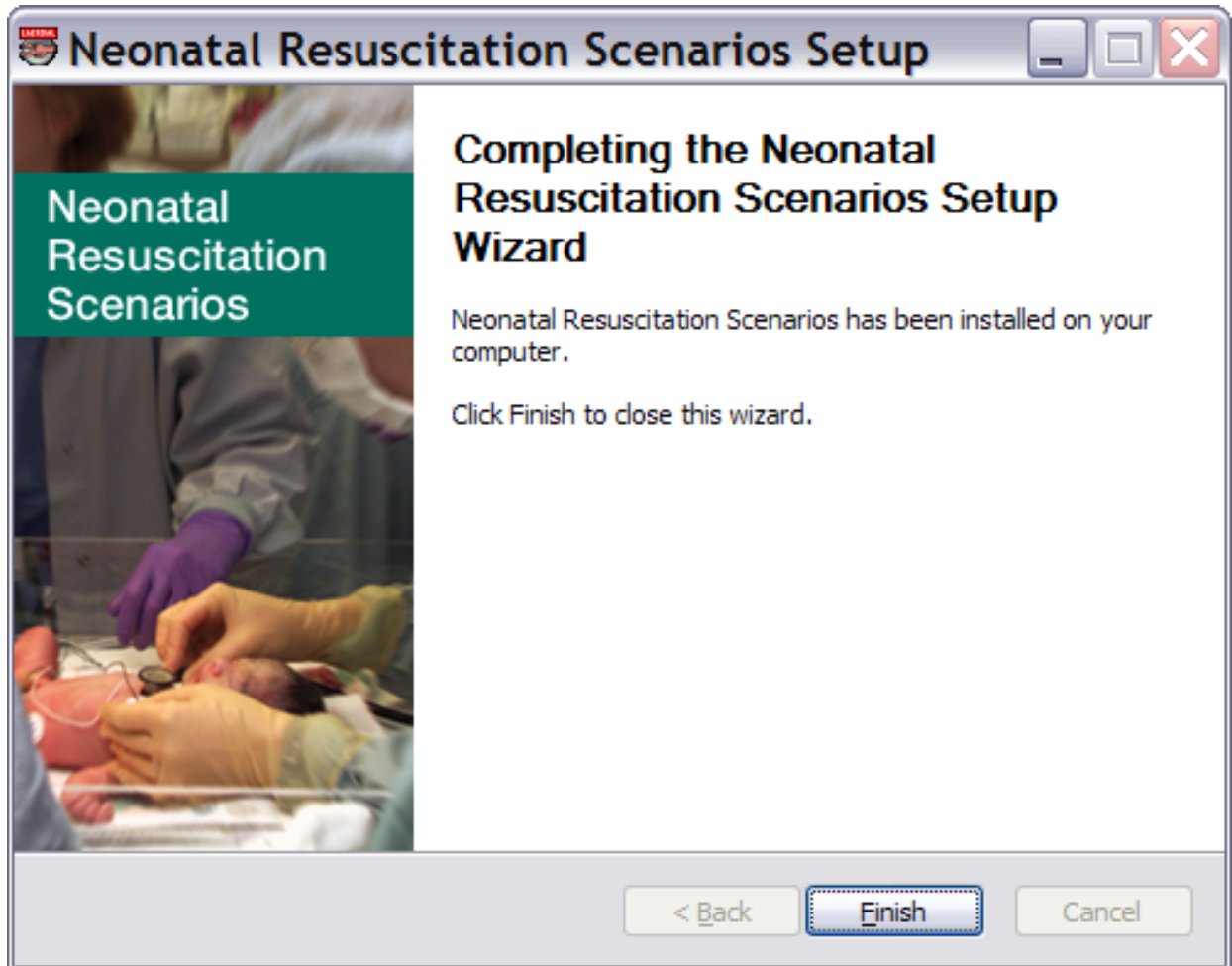

3. Access the updated scenarios in SimNewB

- To get access to the updated scenarios each SimNewB user has to log on to SimNewB first. During the logon process the updated scenario files will be copied to the user's folder.