Laerdal IV Torso

ENGLISH Directions for Use. Page 2 – 3

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
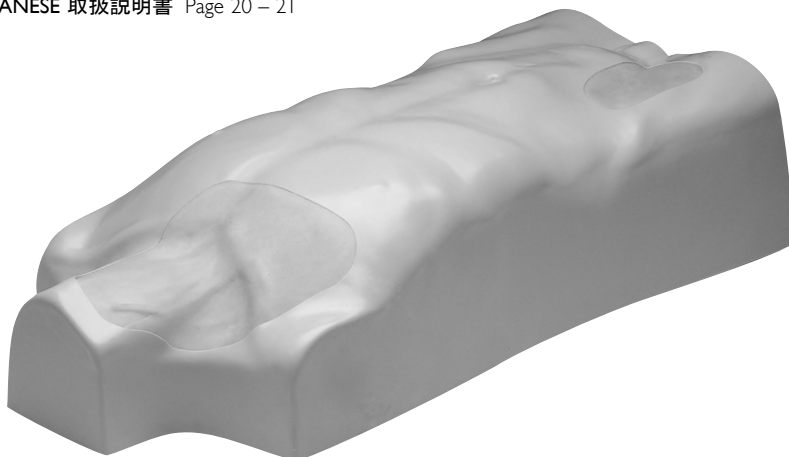
Fig. 1

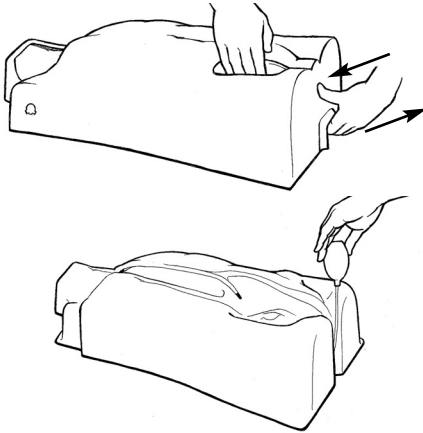Introduction

There are two main IV access areas on the Laerdal IV Torso: The neck/subclavian region and the femoral region. To gain IV access in these two areas, specially designed soft pads can be placed in the recesses in these two areas. The pads include artificial skin, veins pre-filled with simulated blood, palpable arteries, bone and muscle.

Cautions and Warnings

Latex allergy

The replacement pads on the Laerdal IV Torso contain latex. Users who suffer from latex allergy should take precautions while using or handling the replacement pads by wearing protective gloves of a non-latex material.

Cleaning

Do not use alcohols to clean the latex pads. Use water only.

Items included

The Laerdal IV Torso Basic version includes an internal and external shell, pulse tube set, a bottle of simulated blood, one neck replacement pad, one femoral replacement pad, Directions for Use and Carrying Case.

Getting started

Opening the torso shells

Lift the double torso shell from the carrying case. Pull the flap on the waist end of the outer torso shell outwards with your fingers while pressing inwards with your thumb. Lift the waist end upwards until the outer shell comes loose from the inner shell at the head end.

Remove the pulse bulb from its recess in the inner shell. Fig. 1.

Placing and replacing pads

Unpack the Neck pad and Femoral pad and place in the corresponding recesses in the inner torso shell. Position the long veins in the channels in the inner shell. Connect the extending tubes from the pad to the pulse tube at the neck and femoral sites of the inner shell. Fig.2.

Fig. 2


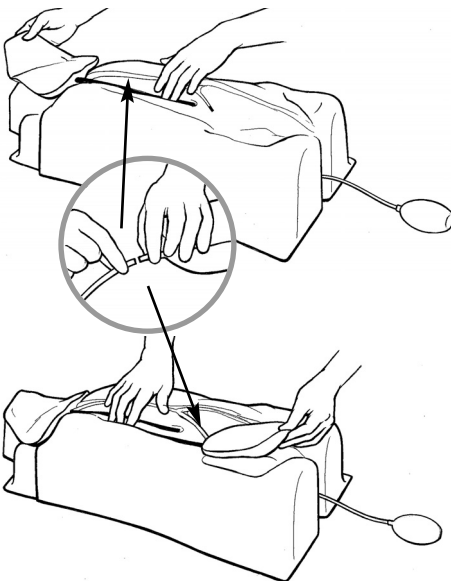
Fig. 3

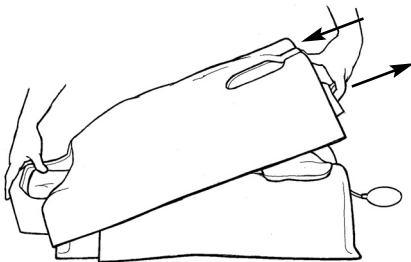The pads are prefilled with blood and can be used for practice immediately after unpacking and assembly. To remove a pad, simply disconnect the pulse tube and lift out the pad.

Closing the torso

Place the head end of the outer shell at an angle down against the head end of the inner shell. At the waist end of the outer shell, pull the flap outwards while pressing inwards with your thumb. Lower the outer shell in place over the inner shell until it fits snugly over the pads. Fig 3.

In Use

Needle recommendations

External Jugular

No larger than 18G

Subclavian, Internal Jugular & Femoral

No larger than 14G

Refilling pads with simulated blood

The artificial skin and veins are designed to self-seal after withdrawal of a needle or catheter. However, since the volume inside the enclosed veins is limited, it is sometimes necessary to replace blood that is lost during practice or when "flash back" is insufficient. A bottle of simulated blood is enclosed with each Laerdal IV Torso. To refill, lift off the outer shell of the torso. Place two catheters into the tip of the long vein extending from the pad. Fill a syringe with simulated blood and inject carefully into one of the catheters, allowing air to pass out of the other catheter until the vein is filled. Be careful not to inject too much blood into the veins at once, since this will cause excessive pressure, leading to leakage of blood out through previous punctures in the veins.

Apart from simple refilling, this technique can be used to simulate various degrees of blood pressure.

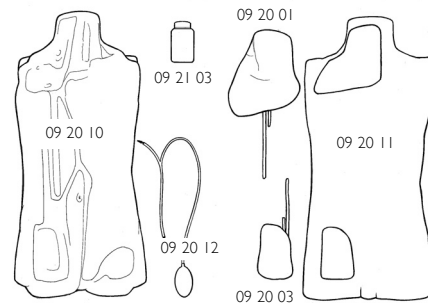
Pulse simulation

The arteries in the soft pads do not contain sim-

ulated blood, only air. When the pads are connected to the pulse tubes on the inner torso shell, the external pulse bulb can be used to simulate a palpable pulse on the soft pads.

Maintenance

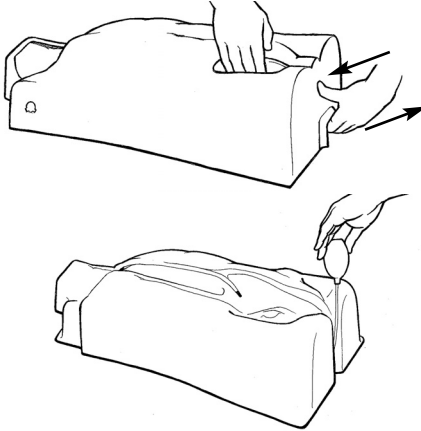
- When withdrawing a catheter from the pad, press a cotton or gauze pad over the point of venipuncture. This will limit the amount of bloodstain on the skin of the pad.
- Use latex or vinyl gloves when handling the IV Torso. This will reduce the risk of staining the pads.
- To remove stains from simulated blood on the pads, wipe the pads in lukewarm water and allow them to dry.

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