# VitalSim™ Arrhythmia Trainer

ENGLISH  
 ESPAÑOL  
 ITALIANO  
 FRANÇAIS  
 DEUTSCH  
 NEDERLANDS  
 NORSK  
 SVENSKA  
 SUOMI  
 CHINESE  
 JAPANESE  
 KOREAN

Directions for Use  
 Instrucciones de Utilización  
 Istruzioni per l'Uso  
 Mode d'Emploi  
 Gebrauchsanweisung  
 Handleiding  
 Bruksanvisning  
 Bruksanvisning  
 Käyttöohje  
 使用说明书  
 取扱説明書  
 사용설명서

**Norway:** LAERDAL MEDICAL AS  
 PO. Box 377, N-4002 Stavanger  
 Tel. +47 51 51 17 00, Fax +47 51 52 35 57  
 E-mail: laerdal.norway@laerdal.no

**Australia:** LAERDAL Pty. Ltd. (ABN 47 003 817 490)  
 23 Edward Street, Oakleigh, Victoria 3166  
 Tel. +61 3 9569 4055, Toll free 1800 331 565  
 Fax +61 3 9569 4028, Toll free 1800 635 835  
 E-mail: customerservice@laerdal.com.au

**Benelux:** "The Laerdal BeNeLux House"  
 Dijkstraat 54, NL-5554 PS VALKENSWAARD  
 Tel. +31 40 208 58 00, Fax +31 40 208 58 02  
 E-mail: info@laerdal.nl

**Benelux:** Belgium and Luxembourg  
 LAERDAL BeNeLux nv  
 Mechelsesteenweg 277, B-1800 VILVOORDE  
 Tel. +32 2 253 36 96, Fax + 32 2 253 36 90  
 E-mail: info@laerdal.be

**Canada and Latin America:**  
 LAERDAL MEDICAL CANADA LTD.  
 151 Nashdene Rd., Unit #45  
 Toronto, ON, Canada, M1V 4C3  
 Tel. +1 (416) 298-9600,  
 Toll free 888/LAERDAL (523-7325)  
 ou en français (800) 567-9987  
 Fax +1 (416) 298-8016  
 E-mail: savelives@laerdal.ca

**Denmark:** LAERDAL DANMARK  
 Esplanaden 18, 1. sal  
 1263 København K  
 Tel. +45 80 333 112, Fax +45 80 333 555  
 E-mail: laerdal.denmark@laerdal.no

**Eastern Europe:** LAERDAL C.E. EUROPE  
 A-1060 Vienna, Mariahilferstr. 1d/1/3  
 Tel-Fax +43 15877140, Tel. +43 15815927  
 E-mail: mstam@aon.at

**Far East:** LAERDAL Singapore Pte Ltd  
 No 1 Marine Parade Central  
 #13-05, Parkway Centre Singapore 449408  
 Tel. +65 63464259, Fax +65 63467523  
 E-mail: admin@laerdal.com.sg

**Finland:** LAERDAL OY  
 Vattuniemenranta 2, 00210 HELSINKI  
 Puhelin +358 (0)9-612 99 80,  
 Fax +358 (0)9-692 77 99  
 E-mail: laerdal.finland@laerdal.no

**France:** LAERDAL MEDICAL FRANCE  
 1 rue des Vergers - Bâtiment n° 5 - 69578 Limonest Cedex  
 Tel. +33 (0)4 72 52 02 52, Fax +33 (0)4 78 35 38 45  
 E-mail: laerdal.france@laerdal.no

**Germany:** LAERDAL MEDICAL DEUTSCHLAND GmbH  
 Am Loserfeld 56, 81249 München  
 Tel. +49 (0)89 / 864 954-0, Fax +49 (0)89 / 864 34 84  
 E-mail: info@laerdal.de

**Italy:** LAERDAL ITALIA s.r.l.  
 Via Piero Gobetti 52/2 int.Z. 40129 Bologna  
 Tel. +39 051-355587, Fax +39 051-355598  
 E-mail: laerdal.italy@laerdal.no

**Japan:** LAERDAL MEDICAL JAPAN K.K.  
 Shinjuku IS Bldg. 8F  
 2-13-12, Shinjuku, Shinjuku-ku, Tokyo 160-0022  
 Tel. 81-3-5919-2121, Fax: 81-3-5919-2122  
 E-mail: laerdal.japan@laerdal.co.jp

**Malaysia:** LAERDAL HOSPILINE Sdn Bhd  
 12 Jalan Titar 33/25, Section 33, Shah Alam Technology Park,  
 40400 Shah Alam, Selangor Darul Ehsan,  
 Tel +60 (3) 5122-7002, Fax +60 (3) 5122-5650  
 E-mail: customerservice@laerdal.com.my

**New Zealand:** LAERDAL NEW ZEALAND LTD  
 (GST Number 79-403-636)  
 PO Box 302664, North Harbour, Auckland 1330  
 Tel. +61 3 9564 4066, Toll free 0800 523 732  
 Fax +61 3 9563 3368, Toll free 0800 528 852  
 E-mail: customerservice@laerdal.co.nz

**Portugal:** LAERDAL ESPANA, Sucursal em Portugal  
 Rua da Bela Vista à Graca, no.31 A, Escritorio 9,  
 1170-054 Lisboa  
 Tel. (+351)-21-816-6567, Fax (+351)-21-816-6568,  
 Mob. (+351)-918765367

**Spain:** LAERDAL ESPANA, S.L.  
 Manuel Tovar 19, 28034 Madrid  
 Tel. (+34) 902-291110, Fax (+34) (91)-7291125  
 E-mail: laerdal.spain@laerdal.no

**Sweden:** LAERDAL MEDICAL AB  
 Box 2064, Paviļongvägen 5, 132 02 Saltjö-Boo  
 Tel. +46 (0)8-55614610, Fax +46 (0)8-55614619  
 E-mail: kontakt@laerdal.se

**United Kingdom:** LAERDAL MEDICAL LTD.  
 Laerdal House, Goodmead Road, Orpington,  
 Kent BR6 0HX  
 Tel. +44 (0)1689 876634, Fax +44 (0)1689 873800  
 E-mail: customerservice@laerdal.co.uk

**USA:**  
 LAERDAL MEDICAL CORPORATION  
 167 Myers Corners Road, PO. Box 1840  
 Wappingers Falls, New York 12590-8840  
 Tel. (800) 431-1055, +1 (845) 297-7770  
 Fax (800) 227-1143, +1 (845) 298-4545  
 E-mail: customerservice@laerdal.com

**LAERDAL TEXAS**  
 PO. Box 38, 226 FM 116 South  
 Gatesville, Texas 76528-0038  
 Tel. (800) 433-5539, (254) 865-7221  
 Fax (254) 865-8011  
 E-mail: customerservice@laerdal.com

This product is in compliance with the essential requirements of Council Directive 89 / 336 / EEC; EMC Directive.

**Laerdal**  
 helping save lives

**Laerdal**  
 helping save lives

**Table of Contents**

Warranty .....3  
 Laerdal Recommends.....3  
 Items Included.....3  
 Skills Taught.....3  
 Cardiac Skills Simulation.....3  
 Safety Precautions .....4

**Language Versions**

Español .....5  
 Italiano .....7  
 Français.....9  
 Deutsch ..... 11  
 Nederlands ..... 13  
 Norsk..... 15  
 Svenska ..... 17  
 Suomi ..... 19  
 中文 ..... 21  
 日本語..... 23  
 ..... 25

**Warranty**

Laerdal Medical warrants to the purchaser that its products are free from defects in material and workmanship for a period of one (1) year from the date of purchase by the original user. During the designated one (1) year period, Laerdal Medical will, upon receipt of a product found to be defective due to materials or workmanship from the purchaser and notification in writing of the defect, at its option repair or replace any parts found to be defective or the entire product. Warranty period does not renew with replacement or repair of original product.

Products found to be defective and notification of defects may also be sent to the authorized Laerdal Medical dealer from whom the product was purchased. All postage, shipping or handling charges shall be the sole responsibility of the purchaser.

Laerdal Medical is responsible for the effects of safety, reliability and performance of its product(s) only if:

- service, repair, readjustment or modification is carried out by Laerdal Medical or persons authorized by Laerdal Medical.
- the electrical installation of the room where the product is used complies with pertinent equipment requirements.
- the product is used in the proper manner in strict compliance with its Directions for Use.

Laerdal Medical shall not be liable under this warranty for incidental or consequential damages, or in the event any unauthorized repairs or modifications have been made or attempted, or when the product, or any part thereof, has been damaged by accident, misuse or abuse. This warranty does not cover batteries, fuses, normal wear and tear, excessive use, staining, discoloration or other cosmetic irregularity which does not impede or degrade product performance.

Some states in the USA do not allow the exclusion or limitation of incidental or consequential damages, so those limitations or exclusions may not apply to you.

There are no other express or implied warranties, whether of merchantability, fitness or purpose, or otherwise, on the product, its parts and accessories.

The VitalSim™ Arrhythmia Trainer is an accessory to the VitalSim, Vital Signs Simulator unit that enables visualization of ECG rhythms on a standard monitor/ defibrillator, pacing with variable capture threshold and defibrillation of up to 360 Joules.

**Laerdal Recommends:**

For use with:

- Standard 3-4 lead ECG monitoring systems.
  - Standard Defibrillator
  - Standard External Pacer
- Not for use with self adhesive defibrillator electrodes.

**Items Included:**

- (1) VitalSim Arrhythmia Trainer\*
- (1) Connector Cable
- (1) Carry Case

\* Must be used with VitalSim Vital Signs Simulator (sold separately)

**Skills Taught:**

- ECG Recognition
- 3 – 4 lead ECG, pacing, with or without capture, and defibrillation (25 – 360 joules)
- Pacing with variable threshold
- Ability to use VitalSim Scenarios

**Cardiac Skills Simulation -**

**Attachment of Arrhythmia Trainer to VitalSim Unit For Cardiac Skills Simulation -**

Connect VitalSim Arrhythmia Trainer to the VitalSim Unit, via cable. (Photo I)

Photo I

For monitoring purposes, lead placement snaps are located on the trainer are as follows: (Photo 2)

- a. RA (Right Arm)
- b. LA (Left Arm)
- c. LL (Left Leg)
- d. RL (Right Leg)

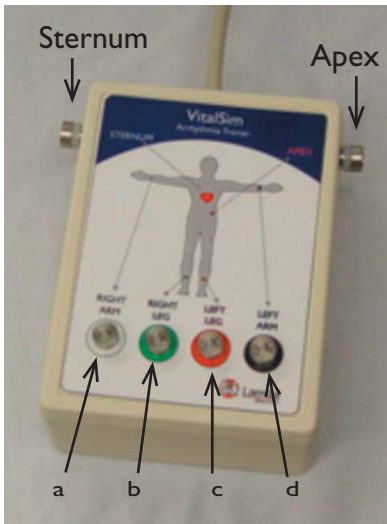
Photo 2

For defibrillation purposes, hands free electrode placement snaps are located on the trainer as follows: (Photo 2)

- Sternum
- Apex

Rhythm may be read from monitor by applying hands free electrodes to sternum and apex locations.

**(Do not use gel.)**

**Laerdal Recommends:**

**Caution:** Follow defibrillation protocol by avoiding contact between the electrodes and any of the ECG sites while defibrillating. A load of up to 360 joules can be delivered.

**Safety Precautions used during Defibrillation of a Patient/Trainer**

1. Read and follow all safety and operating instructions provided with your defibrillator and associated equipment.
2. This trainer can be shocked with actual voltages and current used on a live patient. All precautions and safety measures must be used during the defibrillation and pacing phases of training. Failure to follow safety measures could result in injury or death to operators, students or onlookers not heeding these warnings.

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